## **Noltrex Vet**

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# Synthetic hydrogel for long-lasting joint health

Noltrex®Vet (4.0% Polyacrylamide) is a **highly viscous, synthetic hydrogel for intra-articular injection**. The hydrostatic pressure inside the joint presses the gel up against the inner linings of the joint where it forms a fine, lubricating film. By restoring functional joint lubrication, Noltrex®Vet reduces friction and physically **protects the joint from the effects of overuse which leads to inflammation and pain**.

1

## Proven efficacy

NOLTREX Vet provides effective treatment for horses in various stages of joint disease.

- **Effective for lameness and persistent pain in the affected joint.** NOLTREX Vet is longer lasting than other therapies. due to the unique formulation and action, NOLTREX Vet can be beneficial when other joint therapies have failed.
- **Supportive treatment for cartilage abnormalities and osteochondral defects.** Accumulation in areas of cartilage damage physically protects tissues. Improves lubrication and helps stabilize synovial health to maintain joint function.
- **NOLTREX Vet joint management is compatible with other treatments.** Intra-articular NOLTREX Vet has no interaction with systemic medications. Other therapies do not influence the effectiveness of NOLTREX Vet.

2

## Long-term viscosupplementation

NOLTREX Vet has a protective effect on joint tissues.

- **Restores joint function.** Provides effective viscosupplementation to reduce friction and maintain healthy viscoelastic functionality within the joint. NOLTREX Vet is a physical agent that help restore healthy joint function which secondarily leads to a reduction of pain and inflammation.
- **Immediate protection to cartilage.** NOLTREX Vet acts like a boundary lubricant to reduce friction and protect the cartilage. Forms a surface layer on articular cartilage, even in areas of damage.
- **Prolonged cartilage protection.** Reducing physical microtrauma and surface friction minimizes further cartilage destruction. Repeated injections of NOLTREX Vet long-acting hydrogel are safe and an effective treatment option to many horses.

3

## High safety of application

NOLTREX Vet does not affect natural metabolism in the synovial environment, and undergoes safe excretion from the body.

- **Material is inert and highly compatible with tissues.** NOLTREX Vet has no local and systemic pharmacological, immunological and metabolic effects. The presence of NOLTREX Vet does not change the properties of the natural synovial fluid and does not affect its movement in the joint.
- **Protection of the endoprosthesis.** The polyacrylamide macromolecule is stabilized by 3D bonds, which provides protection against biodegradation by hyaluronidases. The presence of trace amount of silver ions in NOLTREX Vet helps to protect the gel from bacterial colonization\*.
- **Proved safe excretion from the body.** NOLTREX Vet undergoes complete biodegradation through hydrolysis and phagocytosis by synoviocytes. Polyacrylamide is safely eliminated from the synovial environment and the body over time.

\*NOLTREX Vet however does not have any anti-bacterial mode of action and should not be used in presence or risk of infection.

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
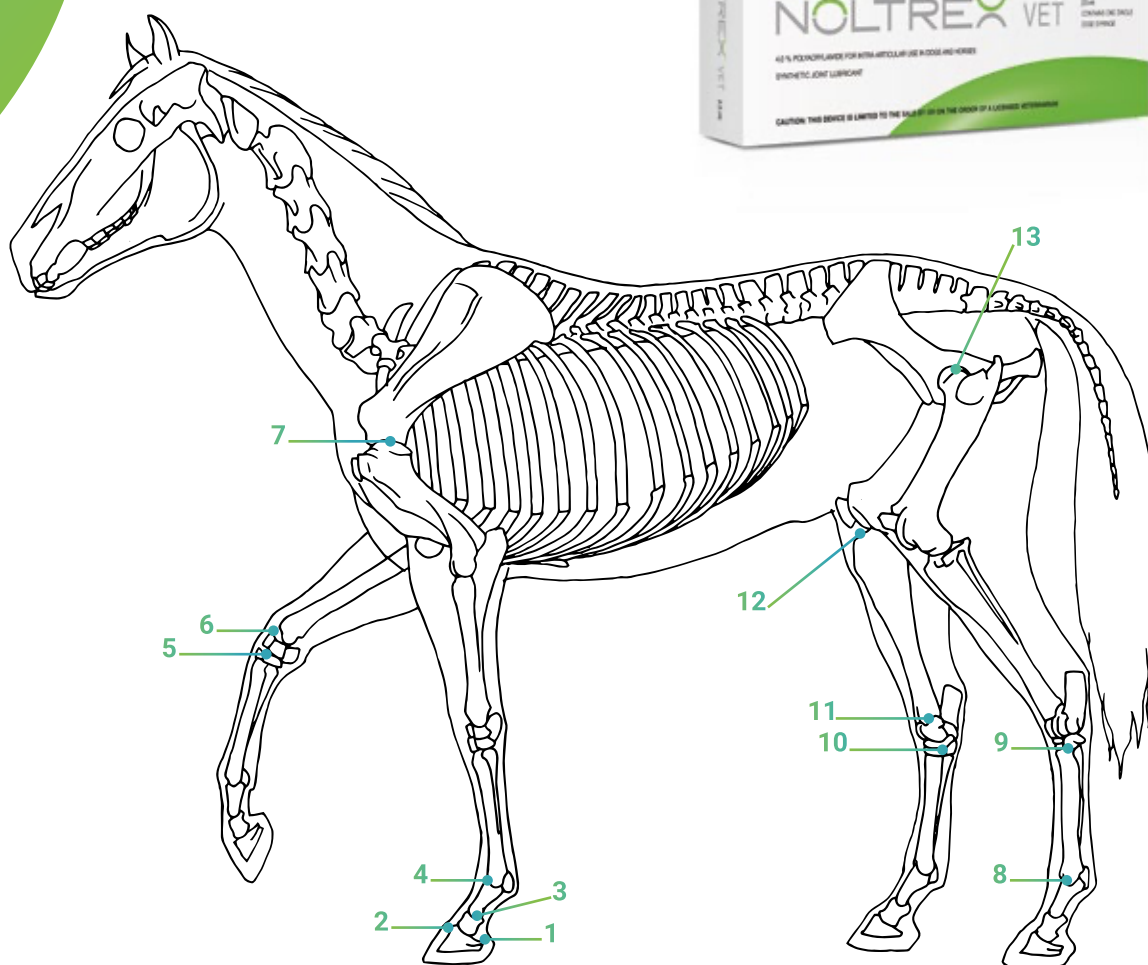
# Noltrex Vet protects the joint for a long time from the effects of overuse.

## Noltrex vet dosages for equines

| General term       | Phalangeal joints                      |                             |                               | Metacarpal phalangeal      | Carpus           |                  | Shoulder            | Metatarsal phalangeal      | Tarsus and distal tarsus |                          |                   | Stifle                          | Coxal joint       |
|--------------------|----------------------------------------|-----------------------------|-------------------------------|----------------------------|------------------|------------------|---------------------|----------------------------|--------------------------|--------------------------|-------------------|---------------------------------|-------------------|
| Veterinarian term  | Hoof heel                              | Coffin joint                | Pastern joint                 | Front fetlock joint        | Knee joint       |                  | Shoulder joint      | Hind fetlock joint         | Hock joint               |                          |                   | Stifle joint                    | Hip joint         |
| Anatomical term    | 1<br>Bursa of the distal sesamoid bone | 2<br>Distal Interphalangeal | 3<br>Proximal Interphalangeal | 4<br>Metacarpal phalangeal | 5<br>Intercarpal | 6<br>Radiocarpal | 7<br>Scapulohumeral | 8<br>Metatarsal phalangeal | 9<br>Tarsometatarsal     | 10<br>Distal Intertarsal | 11<br>Tarsocrural | 12<br>Medial Femorotibial       | 13<br>Coxofemoral |
| NOLTREX dosage, ml | 1.0 - 2.0                              | 1.0 - 2.5                   | 1.0 - 2.0                     | 1.0 - 2.5                  | 1.0 - 2.5        | 1.0 - 2.5        | 2.5                 | 1.0 - 2.5                  | 1.0 - 2.0                | 0.5 - 1.25               | 2.5               | 2.5 - 5.0 depending on severity | 2.5               |

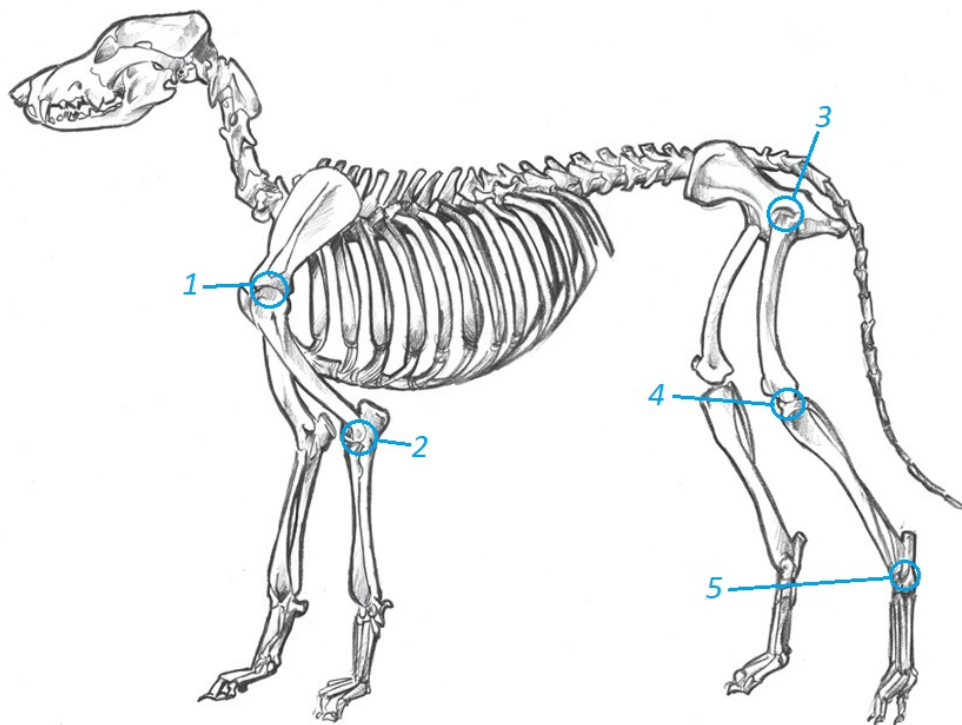\* In chronically affected joints, use a smaller volume and perform a second injection 5 to 6 weeks after initial injection.  
 \* In large joints, perform a second injection 5 to 6 weeks after initial injection.

#### NOLTREX VET DOSAGES FOR DOGS

|              | 5 to 9 kg          |                | 10 to 19 kg |         | 20 to 29 kg |         | 30 to 39 kg |         | 40 to 50 kg |             | over 50 kg |             |
|--------------|--------------------|----------------|-------------|---------|-------------|---------|-------------|---------|-------------|-------------|------------|-------------|
|              | NOLTREX dosage, ml | Needle size, G | Dosage      | Needle  | Dosage      | Needle  | Dosage      | Needle  | Dosage      | Needle      | Dosage     | Needle      |
| (1) Shoulder | 0,3 – 0,4          | 23G 22G        | 0,7 – 0,8   | 22G     | 0,8 – 1,0   | 22G     | 1,3 – 1,5   | 21G 20G | 1,5 – 2,0   | 21G 20G 19G | 2,0 – 3,0  | 21G 20G 19G |
| (2) Ulnar    | 0,3 – 0,5          | 23G 22G        | 0,7 – 0,8   | 22G 21G | 0,8 – 1,0   | 22G 21G | 1,3 – 1,5   | 21G 20G | 1,5 – 2,0   | 21G 20G     | 2,0 – 2,5  | 21G 20G     |
| (3) Coxal    | 0,3 – 0,5          | 23G 22G        | 0,5 – 0,8   | 22G 21G | 0,8 – 1,0   | 22G 21G | 1,3 – 1,5   | 21G 20G | 1,5 – 2,0   | 21G 20G 19G | 2,0 – 2,5  | 21G 20G 19G |
| (4) Knee     | 0,3 – 0,5          | 23G 22G        | 0,5 – 0,8   | 22G 21G | 1,0 – 1,5   | 22G 21G | 1,3 – 1,5   | 21G 20G | 2,0 – 2,5   | 21G 20G 19G | 2,5 – 4,0  | 21G 20G 19G |
| (5) Hock     | 0,2 – 0,3          | 23G 22G        | 0,4 – 0,5   | 22G     | 0,6 – 0,8   | 22G 21G | 1,0 – 1,3   | 21G 20G | 1,0 – 1,3   | 21G 20G     | 1,5 – 1,8  | 21G 20G     |

#### RECOMMENDED NEEDLE GUIDE FOR USE WITH NOLTREX VET

|                             |              |              |              |              |              |                 |
|-----------------------------|--------------|--------------|--------------|--------------|--------------|-----------------|
| External diameter of needle | 23G (0,6 mm) | 22G (0,7 mm) | 21G (0,8 mm) | 20G (0,9 mm) | 19G (1,1 mm) | 18G (1,25 mm) * |
| Cannula color coding        | blue         | black        | green        | yellow       | creamy       | pink            |

\* Not recommended for use in dogs of various breeds. In exceptional cases and only in very large dogs.